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POLYGLYCOLIC ACID (PGA)
coated violet braided / coated undyed braided

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Braided synthetic absorbable suture

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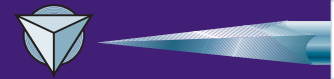
Violeta o Incoloro

TECHNICAL DATA

Polyglycolic Acid (PGA) braid, is endowed with remarkable **strength and suppleness** due to the most recent manufacturing and weaving processes.

The polycaprolactone and calcium stearate-based coating gives **SSA 90** braid the best handling properties for optimum use: after passing through the tissue several times, it remains impeccably smooth.

The **excellent knot security** guarantees the optimum **safety** required by surgeons. The absence of accidental overtightening of knots enables precise positioning of the braid.

SSA 90 is characterised by very high initial strength and a reabsorption, adequated to the healing processes, thus providing the **maximum safety** expected from synthetic absorbable braids. Its violet colour allows a **perfect visibility** on the surgical field.

ABSORPTION

Its mechanism of Absorption takes place by hydrolysis and guarantees a very high rate of tensile strength and a minimum degree of absorption in the initial critical phase of healing. The total absorption duration is between 50-80 days after the implantation, avoiding adverses reactions and tisular inflammation.

TENSILE STRENGTH

The residual tensile strength is the tensile strength that the suture supports during the healing period. After implantation, the strength of an absorbable suture decreases over time. When the residual strength of the braid is zero, a certain amount of non-absorbed material still remains. The retention time represents the time during which the suture retains a certain degree of tensile strength.

TYPE

Synthetic absorbable suture braided and coated

COMPOSITION

Polyglycolic Acid (PGA)

COATING

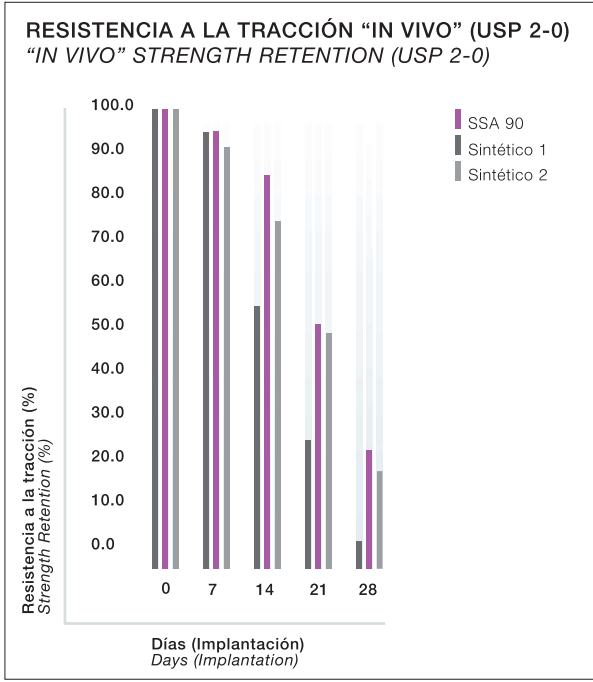
Polycaprolactone and calcium stearate

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
★ All **TAPER NEEDLES** are made from a special steel of a high strength to the deformation (very high bending moment).

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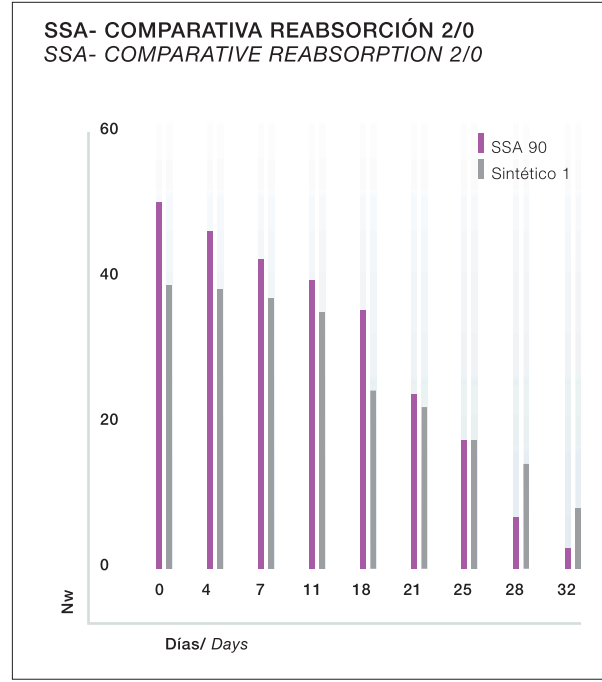The tensile strength of SSA 90 is very high. SSA 90 is effective for 2 to 3 weeks after implantation.

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